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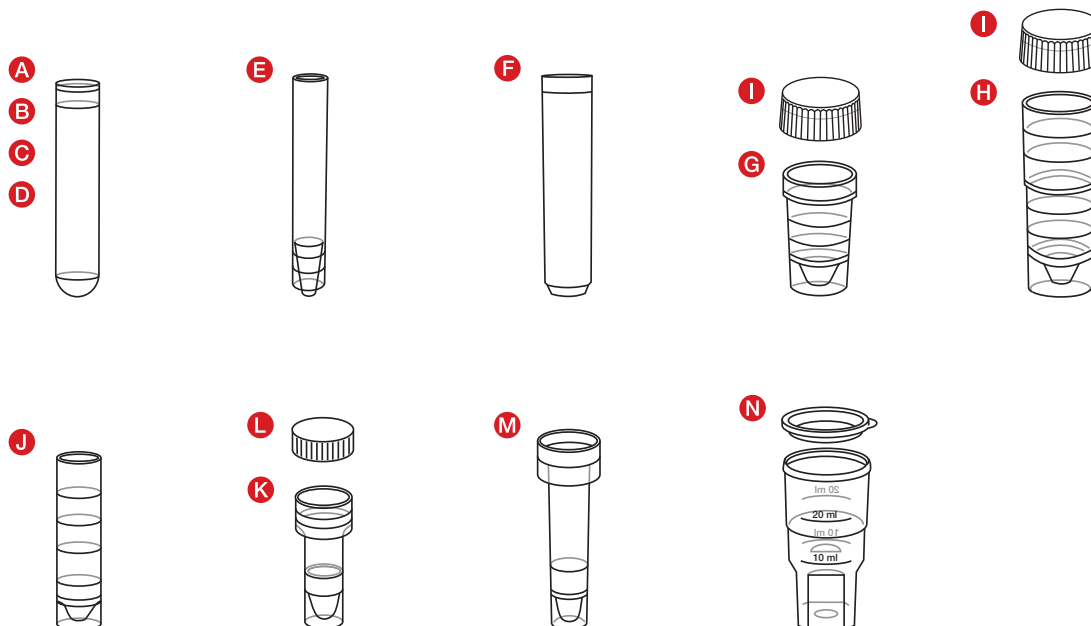
Intended use

The containers for analysis systems are intended for holding specimen material (e.g. serum, plasma or control solutions) for subsequent analysis in the respective analysis systems (see table below). The specimen material in the containers for analysis systems can be sent for automatic or manual specimen processing. The specifications of the device manufacturer must be noted. The products are intended for use in a professional environment by qualified medical and laboratory personnel.

Product description

The containers for analysis systems are available in different variants. The designs vary in capacity, dimensions, shape of base, and various closure options. In addition, sterile and non-sterile versions are available.

Product overview

Containers for analysis systems:

Type	Art. no.	Description
A	55.1578	Tube, 5 ml, 75x12 mm, round base, highly transparent, suitable for FACSCan flow cytometer
B	55.1579.xxx	Tube, 5 ml, 75x12 mm, round base, PS, suitable for flow cytometers such as FACSCanto, FACSAria, FACSCalibur, LRS2, FC500
C	55.476.061	Tube, 5 ml, 75x12 mm, round base, suitable for Vitek and Vitek 2 from bioMérieux
D	55.526.053	Tube, 5 ml, 75x12 mm, round base, PS, suitable for Abbott m1000
E	57.477.500	Tube, 4.5 ml, 75x12 mm, pointed base, for Pharmacia/LKB Gamma Counter and Gamma Counter Cobra from Packard
F	68.752	Round cuvette 3.5 ml, height: 51 mm, suitable for LKB-analyzer and specimen preparation
G	73.641.xxx	Specimen container 24x14 mm for Kone-analyser 73.641.010 for AAS
H	73.646	Specimen container 38x14 mm for autoanalyser
I	65.649	Lid for autoanalyser specimen containers, fits 73.641.xxx, 73.646.xxx
J	73.663	Specimen container 2 ml for Boehringer ES300/600
K	73.666	Specimen container 2 ml, 38x17 mm, for Hitachi autoanalyser
L	65.718	Lid for Hitachi container, fits 73.666
M	73.667	Specimen container 2 ml for Abbott AxSYM
N	73.650.500	Specimen container 60x34 mm for Coulter Counter with enclosed lid

Safety information and warnings

Do not use the product if the packaging is damaged.

1. General precautions: Use gloves and general personal protective equipment to protect yourself from possible exposure to potentially infectious specimen material and transmitted pathogens.
2. Handle all biological specimens and collection utensils according to the guidelines and procedures in your institution.
In case of direct contact with biological specimens, consult a doctor due to the risk of transmission of HIV, HCV, or HBV or other infectious diseases.
Always follow the safety guidelines and procedures of your facility.
3. Check each individual package for damage before use and do not use if packaging is damaged.
4. The containers for analysis systems are intended for single use only. Dispose of all products in the hazardous biological waste containers.
5. Do not use the product after the expiry date. The expiry date of the containers for analysis systems refers to the last day of the month and year as indicated.

Storage

Store the products at room temperature.

Handling

Use gloves, gown, goggles or other suitable protective wear to protect against bloodborne pathogens or potentially infectious materials.

Containers for analysis systems

1. Open the bag and take out the containers for analysis systems.
2. Place your specimen material into the respective container for analysis systems. Make sure that the container for analysis systems is not filled above the nominal volume.
3. For use in devices or automation, follow the device manufacturer's instructions.
4. Close the container for analysis systems when needed with the corresponding cap.

Disposal

1. The general hygiene guidelines and regulations for the proper disposal of infectious material must be observed and complied with.
2. Disposable gloves prevent the risk of infection.
3. Contaminated or filled products must be disposed of in suitable containers for hazardous biological waste that can be subsequently autoclaved and incinerated.
4. They must be disposed of in a suitable incinerator or by autoclaving (steam sterilisation).

Key for symbols and labels:

	Article number
	Batch number
	Use by
	CE marking
	For <i>in-vitro</i> diagnostic use
	Follow the instructions for use
	Risk of contamination if reused
	Keep away from sunlight
	Store in a dry place
	Do not use if package is damaged
	Manufacturer
	Country of manufacture

The following also applies to sterile products:

	Sterilisation by irradiation
	Single sterile barrier system
	Do not resterilise

Technical modifications reserved.

All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority.

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